

# Pictures are Small in Posture Pro

ISSUE: When I import pictures into Posture Pro they only take up about half of the screen.

It's important that you turn the camera sideways when you take the picture. This better frames the patient in the camera. A person is taller than they are wide. Turning the camera sideways creates a better proportioned picture. Now when you import the image it fills up the entire screen.

ISSUE: My tri-pod won't let me turn the camera sideways.

You will have to purchase a tri-pod that does allow you to flip the camera sideways to take pictures in the portrait mode. Or, you can purchase a small one-inch bubble level to hold on the camera (preferred) and you can eliminate the tri-pod altogether.

Online URL: <https://posturepro.phpkb.cloud/article.php?id=14>