

## What are the pink boxes of text on the line behind the body?



You are seeing those measurements because Absolute Measurements is checked under Image options on the main screen. This setting allows Posture Pro to make detailed force calculations. To be accurate, Posture pro must be calibrated. Please see the instruction manual or watch the Absolute Measurements training video at the Video Help Library at [www.posturepro.com /](http://www.posturepro.com/)

The numbers in the pink boxes show the amount of travel and in which direction (front-back) the various anatomical areas have moved in relation to the base point in front of the ankle. These measurements are known as the Absolute Posture Offset, versus the Relative Posture Offset measured by the lines formed on the body.

Online URL: <https://posturepro.phpkb.cloud/article.php?id=156>